

Excel V™

A new approach to treating vascular and benign pigmented conditions

Now men and women alike are making unsightly facial and leg veins a thing of the past. With the most advanced technology, the Excel V laser can safely and effectively treat a wide range of vascular and benign pigmented conditions¹ on men and women of all skin tones.

How does Excel V work?

Excel V is a new laser system for providing treatment of a variety of vascular and benign pigmented skin conditions. Many skin conditions manifest in the form of broken or abnormal blood vessels in the skin, as is the case with rosacea, in which the facial blood vessels dilate excessively causing redness and flushing. Excel V treats vascular conditions by administering a unique high power green laser that is absorbed by the abnormal blood vessels in the skin. The laser heats the abnormal vessels causing the vein walls to collapse and seal shut. Over time, the vessels will no longer be visible. Several treatments may be necessary to achieve the desired result.

¹ After your treatment, the appearance of existing veins will be diminished or completely eliminated. However, the development of new veins cannot be prevented. If your body has an underlying vascular problem, you may need additional treatments. Talk to your physician for additional information.

The logo for Excel V, featuring the word "excel" in a lowercase, sans-serif font, followed by a large, stylized checkmark that forms the letter "V". A small "TM" trademark symbol is positioned at the top right of the checkmark.

Patient Testimonial

Christina had the diffuse redness and tiny broken capillaries on her face treated with the Excel V. These signs of photo-damage appeared about the time she hit 50 and although they were easily covered with make up, Christina didn't want to wear foundation from the moment she woke up until the end of the day.

This was Christina's first laser treatment so she approached it with a bit of trepidation. She was pleasantly surprised by the experience. The Excel V laser pulses felt like small pinches and were quite tolerable. The whole treatment took less than 30 minutes. Immediately post treatment, her face was a little red, but by the next day, the broken capillaries and the diffuse redness were gone.

She was amazed at the effectiveness of the treatment. From morning until night, her skin is now clear and evenly toned – even after a glass of wine or a spicy meal. And, best of all, she no longer needs foundation or cover up!

CUTERA®

WORLD HEADQUARTERS
3240 Bayshore Boulevard, Brisbane, CA 94005
Phone: 1-888-4CUTERA | www.cutera.com

©2013 Cutera, Inc. All rights reserved.
Cutera is a registered trademark of Cutera, Inc.

AP0104 Rev C (01/13)



EXCEL V™

Facial and leg veins are now a thing of the past.

A safe and effective treatment for vascular and benign pigmented lesions.


CUTERA®

Poikiloderma




Photos courtesy of Joely Kaufman, MD

Leg Veins

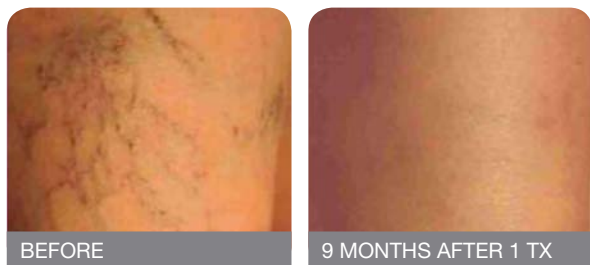

Photos courtesy of Caddell's Laser Clinic

Periorbital Blue Veins

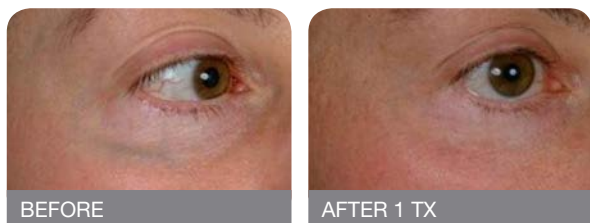

Photos courtesy of Greg Waslen, MD

Surgical Scars



Photos courtesy of Cutera, Inc.

What specific conditions does Excel V treat?

Excel V has been shown effective at treating a number of conditions, including rosacea, spider veins, leg veins, port wine stains, cherry angiomas and benign pigmented lesions such as sun spots. Excel V treats mild and severe forms of these conditions. The system may also have a cosmetic benefit for patients who have skin lesions or scars from past injuries or surgeries.

How is the procedure performed?

After an initial consultation, your physician will determine if treatment with the Excel V is appropriate. Unlike other procedures, injections or topical anesthesia are not required throughout the duration of the 15 – 30 minute procedure. Depending on the condition, the physician passes the Excel V handpiece over the affected areas of the skin until the abnormal vessels diminish². To minimize discomfort and protect the patient's skin, the Excel V uses a proprietary cooling mechanism.

How many treatments are needed?

The number of treatments depends on various parameters, including the size, thickness and color of the vascular lesion. Superficial vascular lesions are usually easiest to treat; some patients might see significant improvement after just one treatment. Since individual results vary, ask your medical professional about your specific condition.

² While some clients may be satisfied after just one treatment, two or more treatments may be needed to achieve desired results.

Do the treatments hurt?

Most patients experience a stinging sensation as the pulses of energy are delivered through the handpiece. Although no local anesthesia or pain medication is typically required, some patients opt to use a topical anesthetic prior to treatment. Following the treatment, the pain is minimal to nonexistent.

What are the possible side effects?

Although many patients report few if any side effects, those most commonly noticed are slight reddening and swelling of the skin. Some patients may experience bruising and, in rare instances, blistering may occur. Consult with your medical professional for complete information on the benefits and risks of treatment.

When will I see results?

Most patients find that the majority of the treated veins have shown significant improvement within two to six weeks of treatment. However, your final results may not be apparent for several months.

How much does an Excel V treatment cost?

Excel V costs varies depending on the size of the area and the condition being treated. Talk with your practitioner to learn more about treatment fees.



For more information on Excel V, please visit www.ExcelV.com